QUARTZ SPLINT®

*Our fiber expertise
is your strength™*

www.rtdental.com

- 
- The background of the advertisement features a large, semi-circular image of a dental splint. The splint is shown in two views: a top-down view of the upper arch and a side view of the lower arch. The splint is made of a clear, translucent material with a fine, grid-like texture. Several fiber-reinforced rods are shown being inserted into the splint. The rods are light-colored with a textured, woven appearance. The overall color scheme is dominated by shades of blue and white.
- GB** INSTRUCTION FOR USE
 - FR** MODE D'EMPLOI
 - ES** INSTRUCCIONES DE USO
 - DE** GEBRAUCHSANWEISUNG
 - GR** ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
 - 中文** 使用说明
 - NO** BRUKSANVISNINGMODE
 - SE** NVÄNDNINGSSINSTRUKTIONER
 - DK** BRUGSANVISNING

NOT5000XX00-201508-31 & NOT5000NO00-201407-16

INSTRUCTIONS FOR USE

DESCRIPTION

QUARTZ SPLINT® is made of quartz fibres and light-cured Bisgma resin matrix. **QUARTZ SPLINT®** is available in different structures: UD for unidirectional (various sizes), **WOVEN** (various sizes), **ROPE** or **MESH** for all directions.

INDICATIONS FOR USE

QUARTZ SPLINT® is used for oral splintings, reinforced frames for bridges or for dentures.

QUARTZ SPLINT® RESIN is indicated to use with Splintings exclusively in case of insufficient resin wetting. It is **NOT** an adhesive on enamel or dentin structures.

QS FLOW™ is a nano filled, light cured flowable composite designed to be used with the **QUARTZ SPLINT®**.

CONTRAINDICATIONS

Allergic reaction to methacrylic resins.

PRECAUTIONS

Avoid to exposure **QUARTZ SPLINT®** which is a light-cured fibre reinforced material to direct light (dental practice or sun)

Keep the products in the original box.

The use of rubber dam is recommended.

ADVERSE REACTIONS

Potential allergic reactions due to methacrylic resins.

STEP-BY-STEP INSTRUCTIONS: DIRECT SPLINTING

1. Isolate tooth and clean the surface.
2. Select the correct **QUARTZ SPLINT® WOVEN** using the table of indications. Determine the length according to the anatomical situation by using a rope or dental floss.
3. Cut the **QUARTZ SPLINT® WOVEN** by using the special scissors and set it back to the

original black box the remaining product. Add **QUARTZ SPLINT® RESIN** if necessary.

4. Etch* and rinse* tooth surfaces.
5. Apply Primer* or Adhesive* (SEALBOND ULTIMA) to tooth surfaces. Gently air dry. Surface should appear shiny.
6. Apply a thin layer of flowable composite* (e.g. QS Flow) along tooth surfaces. Do NOT light-cure at this point.
7. Place and adapt the **QUARTZ SPLINT® WOVEN** against the prepared surfaces. Press the splint on the interproximal surfaces. Light-cured*
8. Cover with a thin layer of composite* (e.g. QS Flow). Light-cure*.
9. Polish with burs, wheels...Avoid exposure the splint when over polishing.

STEP-BY-STEP INSTRUCTIONS: LABORATORY BRIDGE

1. Prepare the tooth with proper chamfer and take an impression.
2. Select the correct **QUARTZ SPLINT® UD** using the table of indications. Determine the length according to the anatomical situation by using a rope or dental floss.
3. Cut the **QUARTZ SPLINT® UD** by using the special scissors and set it back to the original black box the remaining product. Add **QUARTZ SPLINT® RESIN** if necessary.
4. Fill the impression with a layer of the dual-cured composite selected*. Do NOT light-cure.
5. Place and adapt the **QUARTZ SPLINT®** and press slightly the splint into the composite. Light-cure* if necessary.
6. Cover with another layer of composite and press the impression on the dye. Allow to cure.
7. Remove the bridge and finish light-curing*.
8. Refine and polish the bridge.
9. Follow an appropriated procedure to install the bridge.

STEP-BY-STEP INSTRUCTIONS: REPAIR BROKEN DENTURE

1. Grind down the broken parts properly and stick them with stick wax. Prepare the gypsum pad for removable denture
2. Reduce the thickness and width of denture by grinding and isolate the gypsum pad. Determine the length according to the anatomical situation by using a rope or dental floss.
3. Cut the **QUARTZ SPLINT® MESH** by using the special scissors and set it back to the original black box the remaining product.

4. Bond with denture base resin selected*.
5. Place **QUARTZ SPLINT® MESH** and press slightly. Add **QUARTZ SPLINT® RESIN** if necessary. Light-cure 20s with LED or 40s with Halogen lamp by 1cm² area surface.
6. Cover the cured MESH with denture base resin selected*.
7. Polymerize the denture according to the manufacturer instructions*.
8. Refine and polish the denture.
9. Follow an appropriated procedure to install the repaired denture.

STEP-BY-STEP INSTRUCTIONS: PERIODONTAL SPLINTING OF EXTREMELY MOBILE TEETH

1. Isolate tooth and clean the surface.
2. Select the **QUARTZ SPLINT® ROPE** using the table of indications. Determine the length according to the anatomical situation by using a rope or dental floss.
3. Cut the **QUARTZ SPLINT® ROPE** by using the special scissors and set it back to the original black box the remaining product. Add **QUARTZ SPLINT® RESIN** if necessary.
4. Clean*, etch* and rinse* tooth surfaces.
5. Apply Primer* or Adhesive* (SEALBOND ULTIMA) to tooth surfaces. Gently air dry. Surface should appear shiny.
6. Apply a thin layer of flowable composite* (e.g. QS Flow) along tooth surfaces. Do NOT light-cure at this point.
7. Place and adapt the **QUARTZ SPLINT® ROPE** against the prepared surfaces. Press the splint on the interproximal surfaces. Light-cured*
8. Cover with a thin layer of composite* (e.g. QS Flow). Light-cure*.
9. Polish with burs, wheels...Avoid exposure the splint when over polishing

OTHER RECOMMENDATIONS

Do not use the products after the expired date.

SAFETY AND LIABILITY

WARRANTY

RTD warrants this product will be free from defects in material and manufacture. RTD makes no other warranties including any implied warranty of merchantability or fitness for a

particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and RTD's sole obligation shall be repair or replacement of the RTD product.

LIMITATION OF LIABILITY

Except where prohibited by law, RTD will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only.

INDICATIONS

QUARTZ SPLINT® UD (UniDirectional)

Ø 1,0 mm x 80 mm / Ø 1,5 mm x 80 mm

Ø 1,0 mm x 120 mm / Ø 1,5 mm x 120 mm

PERMANENT DENTAL BRIDGES (REINFORCEMENT)

- Laboratory fabrication of composite bridges.

QUARTZ SPLINT® WOVEN

Ø 2,5 mm x 80 mm

Ø 2,5 mm x 120 mm

MOBILE TEETH:

- Splinting of mobile teeth due to periodontal damage

ORTHODONTIA:

- Retention of teeth subsequent to orthodontic therapy

MOBILE TEETH:

- Temporary stabilization of teeth involved in alveolar fractures
- Splinting of mobile teeth due to periodontal damage
- Preventing migration of teeth

QUARTZ SPLINT® ROPE

Ø 1,0 mm x 80 mm

MOBILE TEETH:

- Oral occlusal splints in extremely mobile teeth

QUARTZ SPLINT® MESH

55 mm x 80 mm

REPAIR AND REINFORCE DENTURES:

- Repair of acrylic dentures
- Strengthen acrylic prosthesis during initial fabrication

MANUFACTURER

RTD

3 rue Louis Neel

www.rtd.fr

38120 St Egrève - France

rtd@rtd.fr

** Refer to the manufacturer's instructions for use.*

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